

Helpful Cerner Codes

Edited August 8th, 2020

Order	Command
Overview:	
Routine H&E (green/brown/white)	<ul style="list-style-type: none"> • Add number (can add up to 10 caps)
Embed steps (pink)	<ul style="list-style-type: none"> • “.emb” – embed step sections (choose # of steps, 1 slide/2 slides etc.)
Ribbons (brown)	<ul style="list-style-type: none"> • “.emb”- embed only • “.rib”- ribbons
Decal (white)	<ul style="list-style-type: none"> • “ decal” • Make sure to put hold comment in PTOE on first block of case (example of comment: “A5 held for decal”) • Only have to order once per part of case
Unstained (brown)	<ul style="list-style-type: none"> • “.un”- unstained • Add comment on the number of unstained slides next to the cassette number
Immunoperoxidase/IP's (yellow)	<ul style="list-style-type: none"> • “.ip”- select the type of stain • Can be added to H&E/ribbons/steps
Special package (yellow/brown)	<ul style="list-style-type: none"> • “.pk”- select package • Can be done on H&E/ribbons/steps
Special Stains/SS (yellow)	<ul style="list-style-type: none"> • “.ss” – select special stain that is needed
Sentinel Package (yellow)	<ul style="list-style-type: none"> • “.emb”- embed only • “.pk sent”- select sentinel package
Specific cases	
ECC (green)	<ul style="list-style-type: none"> • Embed H&E
EMC (pink/green)	<ul style="list-style-type: none"> • 1-2 pink caps (.embed steps) • 3 or more caps- Embed H&E
Cones (pink)	<ul style="list-style-type: none"> • “.st4”- 4 cassettes with 4 steps each • For additional cassettes: “.ST1”- 1 cassette with 4 steps
Core biopsies for mass/tumor (brown)	<ul style="list-style-type: none"> • Split into 2 caps, wrap, carbol-fuschin stain • “.pk core” (comment # of unstained slides)
Kidney core biopsy not for mass (yellow)	<ul style="list-style-type: none"> • Kidney package
Liver core biopsy not for mass (yellow)	<ul style="list-style-type: none"> • “.pk liver”- liver package